

CARING FOR YOUR DEVILBISS OXYGEN CONCENTRATOR

DeVilbiss recommends using only original DeVilbiss parts and filters in order to guarantee reliable operation of the product.

WARNING

Do NOT use lubricants, oils or grease.

Before attempting any cleaning procedures, turn the unit “Off.”

Cannula/Mask, Tubing, and Humidifier Bottle

Clean and replace the cannula/mask, tubing, and humidifier bottle according to the manufacturer’s instructions.

Air Filter and Oxygen Outlet Connector

The air filter and connector should be cleaned at least once a week. To clean, follow these steps:

1. Remove the air filter, located on the back of the unit. Remove the oxygen outlet connector (if used).
2. Wash in a solution of warm water and dishwashing detergent (Figure 5).
3. Rinse thoroughly with warm tap water and towel dry. The filter should be completely dry before reinstalling.

CAUTION– To prevent product damage, do not attempt to operate the unit without the air filter or while the filter is still damp.

Exterior Cabinet

As needed, clean the concentrator exterior cabinet by using a damp cloth or sponge with a mild household cleaner and wipe it dry.

WARNING

To avoid electric shock, do not remove the concentrator cabinet. The cabinet should only be removed by a qualified DeVilbiss technician. Do not apply liquid directly to the cabinet or utilize any petroleum-based solvents or cleaning agents.

Use of harsh chemicals (including alcohol) is not recommended. If bactericidal cleaning is required, a non-alcohol based product should be used to avoid inadvertent damage.

