

DeVilbiss iGo® Portable Oxygen System

CARING FOR YOUR DEVILBISS IGO PORTABLE OXYGEN SYSTEM

WARNING

Before attempting any cleaning procedure, turn the unit Off and disconnect from AC or DC power.
Do not use lubricants, oils or grease.

Air Filter

The air filter should be cleaned at least once a week. To clean, follow these steps:

1. Remove the air filter.
2. Wash in a solution of warm water and dishwashing detergent.
3. Rinse thoroughly with warm tap water and towel dry. The filter should be completely dry before reinstalling.

CAUTION–To prevent product damage, do not attempt to operate the unit without the air filter or while the filter is still damp.

NOTE– Device has an internal intake filter that must be inspected by a qualified provider/technician every 3 years.

AC Adapter and Power Cord/DC Adapter

The adapters and power cords should be cleaned as needed. To clean, follow these steps:

1. Disconnect cords from the iGo before cleaning.
2. Clean the cords by using a damp cloth with a mild household cleaner and wipe dry.

Cannula/Mask, Tubing and Humidifier Bottle

Clean and replace the cannula/mask, tubing and humidifier bottle according to the manufacturer's instructions.

DeVilbiss iGo® Portable Oxygen System

Exterior Covers and DeVilbiss Rechargeable Battery

WARNING

To avoid electric shock, do not remove the iGo covers. The covers should only be removed by a qualified DeVilbiss provider. Do not apply liquid directly to the covers or utilize any petroleum-based solvents or cleaning agents.

Use of harsh chemicals (including alcohol) is not recommended. If bactericidal cleaning is required, a non-alcohol based product should be used to avoid inadvertent damage.

The exterior covers and rechargeable battery should be cleaned as needed. To clean, follow these steps:

1. The battery should remain installed and be wiped clean with the covers. If battery is removed, wipe the battery bay and battery with a Dry Cloth only.
2. Clean the iGo exterior covers by using a damp cloth with a mild household cleaner and wipe it dry.

If device has been exposed to contagious pathogens, contact your provider for instructions.

Rolling Carry Case and Detachable Wheeled Cart

The carry case and/or cart should be cleaned as needed. To clean, follow these steps:

1. Remove the iGo from the carry case or cart before cleaning.
2. Clean the case or cart by using a damp cloth with a mild household cleaner and wipe dry.

Battery Storage

Your Battery should be stored in a cool, dry location when not in use.

Battery Disposal / Disposal of Equipment and Accessories

Your Battery is rechargeable and can be recycled. Follow local governing ordinances and recycling plans regarding disposal of device components. Do not dispose of the rechargeable Battery. Contact your DeVilbiss provider before disposal of battery or any iGo components.