

MAINTENANCE / CLEANING

WARNING

- Prior to cleaning, ensure the POC is turned off and unplug any external power sources.
- **DO NOT** disassemble or attempt to repair. There are no user serviceable parts inside. Contact your Equipment Provider for service.
- **DO NOT** spray or apply any cleaners directly onto the case.
- **DO NOT** allow any liquids to drip inside any case openings.
- **DO NOT** use harsh and/or flammable chemicals to clean the device.

Perform cleaning as required.

Cabinet:

1. The patient should connect to an alternate Oxygen source.
2. Select the "OFF" (O) position.
3. Unplug the external power source before cleaning.
4. Clean exterior surfaces of the POC with a cloth dampened with disinfectant solution. *Example: Sporidicid[®] Brand Disinfectant Solution.*
5. Allow POC to air dry.
6. **DO NOT** place inside carry bag or run the POC until it is thoroughly dry.
7. When not in use, store the POC in a clean area free from grease, oil and other sources of contamination.

Inlet Filter:

1. Remove the inlet filter by pulling on the protruding tab.
2. Wash filter with mild detergent. Rinse thoroughly with water and allow to dry completely.
3. Once filter is dry, push the filter back into the case so only the small tab is protruding.

