

SimplyGo Portable Oxygen Concentrator Mobility

Your SimplyGo system has been designed to provide you with a source of oxygen that can be carried with you. To enable you to do so, your SimplyGo device has a carrying case bag, accessory bag, and a mobile cart for use with it.

SimplyGo Carrying Case

The SimplyGo carrying case allows you to take your supply of oxygen with you as you go about your normal, daily activities. It provides protection for your SimplyGo as well as a carrying strap to transport the device. To place the SimplyGo into its carrying bag:

1. Position the bag and the SimplyGo so that the front of the device corresponds with the surface of the bag that has a zippered accessory pouch. This will ensure that the inlet air vent on the rear of the device, the outlet air vent on the front, bottom of the device and the power input connect on the side of the device line up with their openings provided in the bag. If the SimplyGo is not placed in the bag properly the air vents will be blocked, and the device will overheat during operation and shut off.
2. Insert the device into the top of the bag and pull the snugly fitting bag up around the device. Check to ensure that the air vents and power input connector align with their openings in the case.
3. Wrap the handle cover around the carrying handle of the SimplyGo and secure it in place by pressing the two Velcro together.

