

Connecting the Optional Humidifier

Warnings

- Only use the humidifier connector tubing supplied with the SimplyGo Humidifier Connecting Tube kit (REF 1101172) or the SimplyGo Humidifier Pouch and Tube kit (REF 1101602).
- Do not overfill humidifier.
- Do not reverse the oxygen input and output connections. Water from the humidifier bottle will travel through the cannula back to the patient.
- Do not use the humidifier while the device is in Pulse mode.

Note: Only connect the humidifier if prescribed. Only use the humidifier if the device is in Continuous Flow mode.

To connect the humidifier:

1. Attach the humidifier accessory pouch to the mobile cart (see the instructions included with the pouch).
2. Remove the cap from the humidifier bottle.
3. Fill the humidifier with distilled water to the level indicated by the manufacturer.
4. Replace the cap and make sure it is tightened securely.
5. Attach the humidifier bottle adapter tube to the humidifier bottle by turning the wing nut on the bottle counterclockwise until it is fastened securely.
6. Place the humidifier bottle assembly in the accessory pouch (REF 1083699).
7. Rotate the bottle so the humidifier adapter hose is away from the body of the portable oxygen concentrator.
8. Attach the oxygen tubing from the humidifier bottle assembly to the oxygen outlet connector on the portable oxygen concentrator.
9. Attach the cannula tubing to the humidifier bottle outlet.
10. After assembly is complete, make sure the oxygen is flowing through the cannula.

Notes

- A bubble humidifier may be used with the SimplyGo.
- Application accessories shall include a means to reduce the propagation of fire.

- A Humidifier Accessory Pouch (REF 1083699) may be purchased to enable use of a bubble humidifier.

