

### 3. Correct Cleaning and Disinfection Procedure

#### 3.1 How to Clean the Unit after Each Use

##### **Warning**

- Without correct and frequent cleaning and disinfection, micro-organisms may build up in the unit and cause potential infection.
- **Medication cup should be cleaned after each use.**

##### **Caution**


- Keep the battery and power adapter compartment dry at all times.

1. Remove the medication cup from the main unit by pushing medication cup release button and gently pulling up in the direction of the arrow.
2. Open the medication cup cap, empty medication, discard remaining solution and then close cap.
3. Fill the medication cup with distilled water and nebulize 1-2 minutes to remove any residual medication in the mesh holes. Visually inspect the mesh holes immediately.
4. Turn off the device and remove the power source.
5. Remove the medication cup from the main unit.
6. Thoroughly rinse the medication cup (both inside and outside), mesh and mouthpiece with distilled water. Appropriately, scrub the cup and mouthpiece by fingers with distilled water.
7. Air dry the medication cup, mesh and mouthpiece or use a clean cloth or paper towel.



## **Warning**

- Do NOT rinse or immerse the main unit into any kind of liquid.
- Make sure the electrodes are dried.
- Do NOT poke the mesh with finger, cotton swab or any objects.

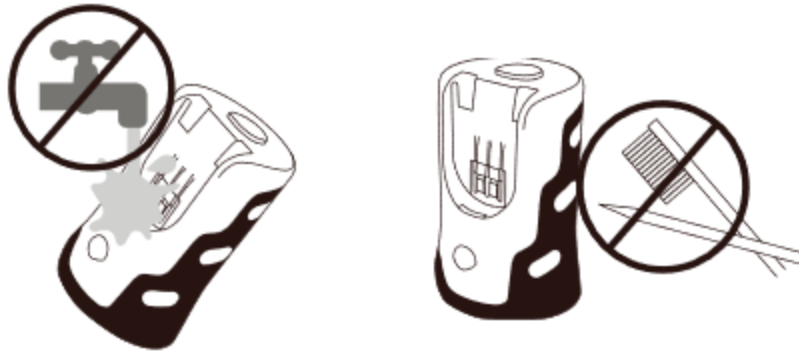

8. Make sure that all cleaned parts are completely dried before installation.
9. Clean the main unit and the electrodes by wiping it off with (moistened) gauze. Use dry gauze to wipe the main unit.

## **Caution**

- Do not use volatile agents such as benzene or alcohol or thinner.
  - Do NOT clean parts in a dishwasher.
  - Do NOT use microwave to dry any parts.
10. Visually inspect the parts after cleaning for any soiling or build-up. If there is any, repeat the cleaning steps.
  11. Assemble the device by following instructions shown Chapter 2.
  12. Store the device in a dry, clean place.

## 3.2 How to Disinfect the Unit

### **Caution**


Without correct and frequent cleaning and disinfection, micro-organisms may build up in the unit and cause potential infection.

### **When to disinfect the unit**

The medication cup, mesh and mouthpiece should be disinfected:

- Before using the device for the first time
- After the unit has not been used for more than a week
- Twice a week during normal use


1. Before disinfection, clean the device by adhering to the protocol shown in Chapter 3.1
2. Bring a large pot of water to a boil and turn off heat. Place medication cup in boiled water for 10 minutes to disinfect.

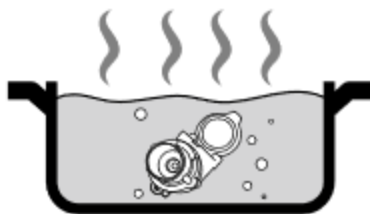

Medication cup

 **Do NOT put main unit in boiled water.**



 **Caution** Be careful of boiled water to avoid being scalded during disinfection process.

3. Remove medication cup from boiled water to lower device temperature.
4. Shake off excess water and air dry on clean cloth or paper towel.
5. Visually inspect the parts after cleaning for any visual soiling or build-up.  
If there is any, repeat the cleaning steps.
6. Store the device in a dry, clean place.